ESCAPE

CHAIR

Instruction videos and more

MOBILITY COMPANY

Manual

Manual - Manuel - Gebrauchsanleitung - Instrucciones - Handleiding

English

Thank you for your purchase! Your Organisation has made the best choice in investing in our evacuation products. Optimize your investment in Escape-Chairs® by taking advantage of training and/or maintenance possibilities. We advise your employees to conduct training in the use of the Escape-Chair®, on a yearly basis. To ensure that your Escape-Chair® remains in optimal order, in the event of an emergency, we advise you to carry out an annual check-up of your equipment. Please contact us for further information on maintenance and/or training possibilities.

Read these operating instructions completely and keep them for reference purposes.

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Warranty

(not including general wear and tear)

- Escape-Chair® : 6 years
- Escape-Carry Chair® : 3 years

Installation and training instructions

These are the instructions for using the Escape-Chair®, as recommended by the Manufacturer.

The evacuation chair is used to help transport persons with reduced mobility.

General:

This manual aims to provide the user with the information required to use the Escape-Chair®, in a safe and correct manner. It contains technical data, care and maintenance instructions, information on spare parts and operation.

The Escape-Chair® may only be operated by trained personnel.

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English

For more information please contact:
Escape Mobility Company Ltd.
85 Great Portland Street, 1st floor,
London, W1W 7LT
T: +44 (0)1908 8229 028/
+44 (0)1908 8229 029
E: info@escape-mobility.co.uk

Warning:

- No alterations may be made to the Escape-Chair®. Escape Mobility Company accepts no liability for improper use or damage caused by unauthorised alterations.
- Always protect yourself against infectious diseases transmitted by blood or other bodily fluids.
- Only use the Escape-Chair® in line with the instructions provided in this manual.
- Only one operator is required to use the Escape-Chair® for evacuation down the stairs. The Escape-Carry Chair® is designed so that a person can be 'carried' up the stairs. For this purpose two or more operators are necessary.

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- Under certain circumstances (stairs covered with carpet flooring, rubber or plastic edging) the Escape-Chair® may leave marks on the stair nosings. However, the braking effect of the Escape-Chair® continues to be optimal.
- Always remain within the maximum weight specification shown in this manual on page 2.
- For the transportation of people heavier than yourself it is important to be aware of the restrictions and possibilities. Trained persons can assess such evacuation.
- If stairs are steep ($\geq 45^\circ$) it is essential that someone should precede the Escape-Chair® during the descent.
- Over time, the glidebelts of the evacuation chair may become dry. Therefore, we highly recommend the requirement of a maintenance contract, so that they can be controlled on a yearly basis.

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Tüv NORD tested up to 275kg and certified up to 185kg. SGS tested and certified up to 225kg.

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**Technical specifications/
List of spare parts:**

1. arm rest*
2. stand with castor wheels
3. castor wheels
4. closed glidebelt cassette
5. large wheels
6. foot rest*
7. seat, with deployment bag
8. head rest
9. spring lock for the operation handle
10. ergonomic back rest
11. operation handle as option two-height adjustable
12. anti-slip foam on operation handle and lower frame bracket*
13. maintenance sticker (which is located on the glide belt holder - 4 - mentions the type of glide belt as well as the date of the most recent inspection)

Attention! Training and maintenance may only be carried out by persons who are qualified by Escape Mobility Company accordingly.

* Not all Models are supplied with arms rests, footrest and/or anti-slip

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Serial no.:	Delivery date:	Belt type standard / standard-
Service date:		
Belts changed on:		

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Mounting the Escape-Chair®:

With every Escape-Chair® there are two wall hooks included. For these to be fixed to the wall, the following is required:

- 6 mm drill
- 4 rawl plugs (supplied)
- 4 x 25 mm screws (supplied)

Position the hooks approx. 250 mm apart and at least 680 mm from the floor. Once the hooks have been mounted, the Escape-Chair® can be mounted with the glide-belts facing towards the wall.

The location of the Escape-Chair®:

We suggest that you place the Escape-Chair® in a strategic place within the Department.

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- foret de 6 mm
- 4 chevilles (fournies)
- 4 x vis 25 mm (fournies)

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Cleaning the Escape-Chair®:

The Escape-Chair® should be kept clean at all times. The use of a dust cover (optional extra) is recommended. If the evacuation chair comes into contact with bodily fluids (e.g. if the evacuee is injured) the Escape-Chair® should be thoroughly cleaned with disinfectant.

Folding out the Escape-Chair®:

1. Remove the Escape-Chair® from the wall hooks and place it in front of you so that you can see the kickstand with the wheels. Place your foot on the lower frame (*refer to pict. 1*).

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2. Take hold of the evacuation chair with both hands on the upper sides of the handle. Pull the handle upwards until you hear the springs click: the extending handle is now locked into position (refer to pict. 2).
3. Slide the headrest downwards.
4. The Escape-Chair® is held together at the back by means of a safety belt. The safety belt can be released by pushing on the sides of the buckle. Now, pull one of the glide-belt holders towards you. This will enable the seat to fold outwards.

Transporting persons:

1. Allow the evacuee to get into the chair. Do not remove your foot from the lower frame before the person is seated (refer to pict. 3).

In case of an evacuation of wheelchair users please place the Escape-Chair® at the left or right side of the wheelchair so that the wheelchair user can sit down in the Escape-Chair® with or without your assistance.

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English

- Place your foot on the kickstand, just above the small wheels and pull the kickstand backwards. The gas strut makes this operation easy. The Escape-Chair® is now completely unfolded and ready for use (refer to pict. 4).
- Fasten the safety belt.
- For the comfort model - push down the arm rests and the foot rest.
- Take hold of the Escape-Chair® keeping your hands at the sides of the handle and walk towards the (emergency) staircase.
- When you arrive at the (emergency) staircase (before you reach the top step) use your foot to push the kickstand with the small wheels inwards, so that it clicks into place (refer to pict. 5).
NEVER USE YOUR HAND.

IMPORTANT: Please take note that failing to fold in the kickstand before descending the stairs, may result in serious injury being sustained to the evacuee.

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7. Balance the chair and push it towards the top of the stairs. Keeping your hands on the sides of the handle, push the evacuation chair onto the first two steps. As soon as the glide-belts touch the first two steps, you will feel the evacuation chair brake. Once the chair is resting on the two steps, move your hands from the sides to the top of the handle. You can now slide down at the same angle as the stairs (*refer to pict. 6*).

8. Keep your hands together and push in the direction of your feet. In this way you can determine the speed of your descent. If you begin to go too fast, use your own weight as a brake by leaning your body backwards. Always keep your hands on the top of the handle when you move down the stairs (*refer to pict. 7*).

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English

9. As soon as the wheels reach the landing you should stop. At this point you must return your hands to the position at the sides of the handle. Balance the chair and use the large wheels to move to the next part of the stairs (*refer to pict. 8*).
10. For steep stairs, we suggest that you work with two operators.
11. When you arrive at the bottom, balance the Escape-Chair® and push it along a short distance, on the large wheels. Unfold the kickstand with the gas strut and push the chair and evacuee to the assembly point (*refer to pict. 9*).
IMPORTANT: Keep the chair balanced at the bottom of the stairs.
12. Never allow the chair to roll forward when you reach the bottom of the stairs.

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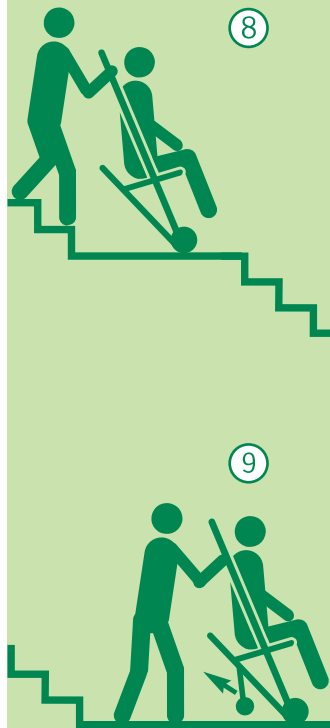
- Align weight directly above the axis.
- Fold the kickstand with wheels, by placing the foot on the middle pedal of the kickstand, until the kickstand 'clicks' back into position (refer to pict. 10).
NEVER USE YOUR HAND.
It might be necessary that additional persons should help the wheelchair user when he leaves his chair.
- Place your foot on the lower frame and push the chair into an upright position (refer to pict. 11).

Folding up the Escape-Chair®:

- Once the kickstand with wheels is folded inwards, place your foot on the lower frame.
- You can rest the Escape-Chair® on your knee. In this way both your hands will be free (refer to pict. 12).

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English

- Using both hands, pull up the head rest.
- Push the springs in the handle inwards simultaneously. The handle will now automatically slide downwards (refer to pict. 13).
- Place the ends of the safety belt inside the seat frame.
- Pull the seat frame towards you, so that it folds back into its original position.
- Take hold of the ends of the safety belt and fold them around the back of the Escape-Chair®, making sure that the glide-belt holders are also secured within the safety belt (refer to pict. 14).
- Fasten safety belt securely.
- You can now return the Escape-Chair® to its original location or use it to evacuate another person.

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Manual - Manuel - Gebrauchsanleitung - Instrucciones - Handleiding

Manual Escape-Carry Chair®

The Escape-Carry Chair® differs slightly from the Escape-Chair®. As well as transportation down the stairs, by using the glidebelts, as described in the manual of the Escape-Chair®, the Escape-Carry Chair® is provided with carrying handles on the upper and lower frame.

When using the carrying handles of the Escape-Carry Chair®, at least two helpers are required. Lower the operation handle back into its original (foldaway) position. (refer to pict. 1). Refer to page 12, point 4.

Unfold the upper (A) and lower (B) handle when using the Escape-Carry Chair® to carry a person. If present, unfold the armrests and footrest.

The upper handle (A) is situated at the back of the Escape-Carry Chair® behind the backrest. This handle is easily pulled upwards, so that it is instantly ready for use (refer to pict. 1 on page 15).

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By means of the solid support structure, the carrying capacity is guaranteed.

When the upper handle (refer to pict. 1) is no longer needed, you can fold it back into its original position.

The lower handles can be set into two adjustable positions (refer to pict. 2 and 3).

Pull the spring clip towards you, in order to change to another position (refer to pict. 2).

By releasing the spring clip, this automatically clicks into one of the two positions. If you want to change to another position, pull the spring clip and move the handle into the other position.

By pulling the spring clip towards you and turning the clip to the left or right, the spring clip will remain in a neutral position. When the handles are no longer needed, fold the handles back into their original position.

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English

Make sure that the centre of gravity of the chair is in the direction of the stairs. Meaning that the persons back is always facing towards the stairs. The helpers, of whom are holding the carrying handles will - depending on the steepness of the stairs - walk forward or backwards (refer to pict. 1, 2, 3 and 4).*

When the stairs are wide enough to work with four helpers, they can be positioned on either side of the Escape-Carry Chair®. Both carrying handles can be operated by two helpers per handle. The helpers decide as to which is the safest way in which to ascend the stairs.

* Attention:
The way in which the Escape-Carry Chair® is used will depend on the situation. You will need to assess the situation yourself, as to whether the advised actions described will suit the current situation. The skills of the helpers will be of paramount importance.

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